MeniCare Soft

CB MULTIPURPOSE SOLUTION for all soft contact lenses

INDICATIONS

MeniCare Soft is indicated for disinfection, cleaning, protein removal, rinsing, storage of all soft contact lenses.

DIRECTIONS

- Always wash and rinse your hands before handling your contact lenses.
- Always handle the same lens, right or left, first in order to avoid confusion.
- After removing the lens from your eye, place the lens in the palm of your hand.
- Apply a few drops of MeniCare Soft solution to the lens. Rub the lens gently for 20 seconds using your forefinger.
- Rinse the lens completely free from debris with fresh MeniCare Soft solution.
- Place the lenses in the right and the left lens case filled with MeniCare Soft solution and soak them at least 4 hours for disinfection (or overnight).
- Your lenses are now clean and disinfected.
- Before wear, rinse your lenses with MeniCare Soft solution.
- Pour the MeniCare Soft solution out of the lens case and rinse the case with fresh MeniCare Soft solution. Leave the case with the caps off to dry in the air. Place back the caps. Your case is now ready for use again.

Lenses may be stored in the unopened case until ready to wear, up to a maximum of 30 days. If you store your lenses for longer periods of time, they must be cleaned and disinfected with fresh MeniCare Soft solution every 30 days.

WARNINGS

- Always use fresh MeniCare Soft solution – never re-use or “top-off” old solution in your lens case since solution reuse reduces effective lens disinfection and could lead to serious eye infection.
- Never use tap water or non-sterile water with contact lenses as microbial contamination can occur, which may lead to permanent eye damage.
- Keep out of reach of children.
- Consult your doctor or eye care professional before using any eye medication with contact lenses.

PRECAUTIONS

- Do not use after expiry date. 
- Do not use if the tamper-evident ring is damaged.
- Once bottle is open, discard any remaining solution after 3 months.
- Close bottle after use.
- Do not touch the bottle top with fingers.
- Store below 25°C. 
- Store out of direct sun light. 
- Do not swallow.
- Keep the lens case clean and dry; renew it regularly.
- Consult your eye care practitioner before changing solutions.
- Do not use for Rigid Gas Permeable contact lenses.

CONTRAINDICATIONS

People allergic to any ingredient with MeniCare Soft Solution should not use this product. Never use MeniCare Soft Solution for heat disinfection.

ADVERSE REACTIONS

If you experience persistent discomfort or irritation, remove your lenses, stop using the product and consult your eye care professional.

CONTENTS

MeniCare Soft is an isotonic, aqueous solution containing polyhexanide 0.0001% as the active ingredient and preservative, macrogolglycerol hydroxystearate 60, disodium edetate and isotonic agents.

PACKAGING

MeniCare Soft is available in 50 mL and 360 mL bottles, including disposable lens case.

Manufacturer: 	Distributors:	European Representative: 
Menicon Co., Ltd Nagoya, 460-0006, Japan	Menicon Europe 104, rue Martre-92583 Clichy cedex France Menicon GmbH Odenwaldring 44, D-63069 - OFFENBACH NKL Contactlencen Postbus 47 - 7800 AA EMMEN	Menicon Holdings Europe 104, rue Martre-92583 Clichy cedex France
		

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